SORBACT® RIBBON GAUZE

EN – Instructions for Use

SV – Bruksanvisning

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DA – Brugsanvisning

NO – Bruksanvisning

ET – Kasutusjuhised

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SK – Návod na použitie

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AU/NZ Bayport Brands Pty Ltd, 21 Bayport Court, Mornington, Vic 3931, Australia

LB Food and Drug Corporation Sal, Byblos Bank building, Rachid Karameh street, Verdun, Beirut P.O. Box 11-8182, Lebanon, Tel. +961-1-862000, www.fdc.com.lb

IR NIBYKAN Golden Pharma Mannheim GmbH, Augustaanlage 32, 68165 Mannheim, Germany, Tel. +49 621 43179339

Payk Daru Tosseh, No. 03, Karimkhanzand Ave, 7th of Tir Sq, Tehran, Iran. 1585917114, Tel. +98 21 54549000

Device Description Sorbact® Ribbon Gauze is a sterile, bacteria and fungi binding wound dressing. It consists of a green Sorbact® wound contact layer, which allows passage of wound exudate into a secondary dressing. The dressing can be used in compression therapy.

Mode of Action Sorbact® binds microorganisms, such as *Staphylococcus aureus* (including MRSA), *Streptococcus species*, *Escherichia coli*, *Pseudomonas aeruginosa* and *Candida albicans*, as shown *in vitro*. These microorganisms are removed from the wound each time the dressing is changed.

Intended Use Sorbact® Ribbon Gauze is intended for use in management of clean, contaminated, colonized or infected exuding wounds, such as cavity wounds and fistulas. Sorbact® Ribbon Gauze is also intended for use in fungal infections in skin folds.

Contraindications There are no known contraindications to the use of Sorbact® Ribbon Gauze. Attention is drawn to the warnings and precautions.

Warnings and Precautions

If the treated condition deteriorates, fails to improve or a side effect is observed, consult a physician or an appropriate health care provider.

Do not use on patients with a known hypersensitivity to any of the dressing components.

The dressing should always be applied in direct contact with the wound area to allow microorganisms to bind to the dressing.

Should the dressing adhere to the wound, moisten or soak the dressing to assist removal and avoid disruption of the healing wound.

Do not use in combination with fatty products, such as ointments, creams and solutions as they may decrease the binding of microorganisms.

Sorbact® Ribbon Gauze is for single use only and should not be re-used. Re-use may lead to increased risk of infection or cross contamination.

Do not re-sterilize.

Do not use if peel pouch is damaged.

Prior to radiation therapy remove Sorbact® Ribbon Gauze. A new dressing can be applied following treatment.

Sorbact® Ribbon Gauze does not contain x-ray thread and should not be used in open surgery.

Sorbact® Ribbon Gauze is MR safe.

Instructions for Use (wound) **1.** Prepare the wound according to local clinical practice. **2.** Select an appropriate dressing size for the wound. The dressing may overlap the wound margins if needed. **3.** Remove the dressing from the pouch using an aseptic technique. **4.** If cutting the dressing, cut across using an aseptic technique. Discard any open and unused dressing. **5.** Apply the dressing. Ensure that the dressing comes into direct contact with the complete wound surface. **6.** Line/fill the cavity or fistula. Avoid dense packing. Always leave a part of the dressing sticking out so that it can be easily removed. If cutting the dressing it is recommended to leave the cut end outside the cavity and fistula to avoid residual threads in the wound. **7.** Apply an appropriate secondary dressing. **8.** The dressing change frequency depends on exudate levels and the overall condition of the wound and surrounding skin. Should the clinical condition allow, the dressing can be left in place for up to 7 days.

Instructions for Use (fungi) **1.** Prepare the affected area according to local clinical practice. **2.** Do not use antifungal creams or ointments in combination with the dressing. **3.** Select an appropriate dressing size to be able to cover the affected area. **4.** Apply the dressing, cut if necessary. Ensure that the dressing comes into direct contact with the affected area. **5.** If fixation is required a bra or underpants can be used. The use of a skin friendly adhesive is recommended if additional fixation is needed. The adhesive should be applied outside the affected area. **6.** Change the dressing once daily in conjunction with inspection. **7.** Continue to use the dressing for 2-3 days after the symptoms have resolved to avoid recurrence.

Storage and Disposal Keep dry and away from sunlight. Disposal should be made according to local environmental procedures.

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